DE Paraffinbad Gebrauchsanweisung	2	TR Parafin banyosu Kullanma talimatı.....	51
EN Paraffin bath Instructions for use	12	RU Парафиновая ванночка Инструкция по применению	60
FR Bain de paraffine Mode d'emploi	21	PL Wanienka do parafiny Instrukcja obsługi.....	69
ES Baño de parafina Instrucciones de uso.....	31		
IT Bagno di paraffina Istruzioni per l'uso.....	41		

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Warning

- The device is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will invalidate the warranty.

Dear Customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, air and baby.

With kind regards,
Your Beurer team

Getting to know your instrument

The MP 70 paraffin bath brings paraffin wax to melting point. Dry and chapped hands, feet or elbows are submerged repeatedly for short periods of time into the warm liquid paraffin wax and then removed again. This causes a thin layer of wax to form around the affected part of the body, over which a sheet is additionally placed. The warmth of the paraffin wax opens the pores of the skin and moisture escapes. The layer of wax prevents the moisture from escaping. Instead, the moisture enters the opened pores and penetrates deep into the skin, where it is absorbed. This makes skin especially soft and smooth. After an exposure time of 20 minutes, the paraffin wax can be removed from the affected body parts. In addition, the warmth of the paraffin bath improves circulation.

Contents

1. Included in delivery.....	13	8. Cleaning and maintenance.....	18
2. Signs and symbols	13	9. Accessories and replacement parts	19
3. Intended use.....	14	10. What if there are problems?	19
4. Warnings and safety notes.....	14	11. Disposal.....	19
5. Device description.....	15	12. Technical specifications	20
6. Initial use	16	13. Warranty / Service	20
7. Usage	17		

1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

1 x MP 70 paraffin bath

1 x lid

1 x protective grille

2 x 450 g bags of paraffin wax

30 x plastic sleeves

2. Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device:

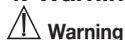
	Warning Warning instruction indicating a risk of injury or damage to health
	Important Safety note indicating possible damage to the device/accessory
	Product information Note on important information
	Read the instructions
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
	Protection class II device The device is double-protected and thus corresponds to protection class 2
	For indoor use only
	Dispose of packaging in an environmentally friendly manner
	CE labelling This product satisfies the requirements of the applicable European and national directives.

	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.
	United Kingdom Conformity Assessed Mark

3. Intended use

This device is intended exclusively for use on the hands, feet or elbows of humans.
Do not use the device on animals!

4. Warnings and safety notes

Warning

- The device is intended solely for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use. This device is not suitable for commercial use or for use in the medical domain. Only use the device in the private domain.
- Repairs may only be conducted by authorised service centres, as special tools and/or parts are necessary for repair, otherwise the guarantee is invalidated.
- Do not use the device if you have heart disease, flushed skin, inflammations, swelling, sensitive areas, diabetes, open wounds, burns, eczema, psoriasis or reduced sensitivity to pain.
- Do not use the device if you are not sensitive to heat.
- If you are pregnant or suffer from health problems, please consult your doctor before using the device.
- If you suffer from acute inflammations or a peripheral vascular disease, please consult your doctor before using the device.
- Never operate the paraffin bath without paraffin wax.
- Remove jewellery from your hands and feet before use.
- Never heat the paraffin wax in its plastic packaging. Never heat the paraffin wax on the hob, over a naked flame or in a microwave. Only heat the paraffin wax in the paraffin bath.
- Children do not recognise the danger posed by electrical devices. Make sure that the device cannot be used by children without supervision.
- Check that the device and the mains cable are in perfect working order before each usage. In the event of damage, the device must not be used.
- Before connecting the device to the mains, check whether the local mains voltage corresponds to the voltage specified on the device.
- Do not use the device near water (e.g. bath, wash basin etc.).
- Do not use the device while bathing, showering or sleeping.
- Never place the device in water or other liquids. Store the device in a safe place.
- Never place the device on easily damaged surfaces.
- When using the device, ensure that it is placed on a stable and level surface.
- Keep the device and mains cable away from hot surfaces. Do not place it on top of other devices, as the hot paraffin wax may cause damage.
- Do not use the device in the vicinity of aerosol products or equipment that emits oxygen.
- Do not use the device outside. The device is only to be used inside.
- Before use, ensure that the mains cable does not pose a trip hazard.
- Do not use the device in bed, whilst sleeping or if you are tired.
- Never use the device under a blanket or towel etc.
- Only use the device with the supplied accessories.
- Do not overfill the paraffin bath. Ensure that the liquid paraffin wax does not exceed the MAX fill indicator line.
- If the paraffin wax has completely melted, the device may be very hot.

- Before submerging your hands, feet or elbows in the liquid paraffin wax, use the inside of your wrist to check that the temperature of the wax is comfortable.

Warning

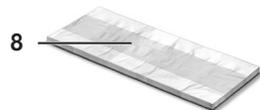
- Should a fault occur during use, switch off the device immediately and remove the mains plug from the socket.
- Keep the hot paraffin wax away from your eyes and other body parts that are sensitive to heat.
- If the paraffin wax permanently feels too hot or too cold against your skin, this may be a physical problem. If this is the case, consult your doctor.
- Make sure that the device is always filled with paraffin wax up to the MIN fill indicator line during use.
- Do not apply the paraffin wax to the treatment area for longer than 20 minutes.
- Do not shake or move the device during use. You could sustain scald or burn injuries.
- If there is hot paraffin wax in the paraffin bath, do not pour in any water.
- If pain, irritation, dermatitis or discomfort occurs, terminate the treatment and consult your doctor.
- Should the device accidentally fall into water, disconnect the mains plug immediately. Have the device checked by a qualified electrician before it is used again.

Important

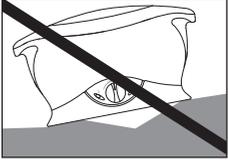
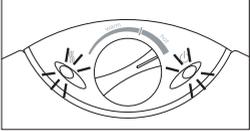
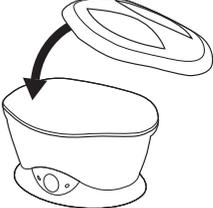
- Switch off the device after use by removing the mains plug from the socket.
- Never remove the mains plug from the socket by pulling on the cable or with damp hands.
- Do not use any sharp or pointed objects to remove the wax block from the paraffin bath. You could damage or scratch the paraffin bath.
- If you are not going to use the device for an extended period of time, store it in a cool, dry and dust-proof place.
- If you should have any questions about using our devices, please contact either your retailer or Customer Services.

5. Device description

1. Paraffin bath
2. Lid
3. Green operation light
4. Temperature controller
5. Red heating light
6. Protective grille
7. Paraffin wax
8. Plastic sleeves

6. Initial use

<ul style="list-style-type: none">• Remove the packaging material.• Make sure that all parts specified in the Included in delivery list are contained (see 1. Included in delivery).• Check the paraffin bath and mains cable for damage.• Place the paraffin bath on a flat, secure, even and water-resistant surface or table.• Make sure that the paraffin bath is empty before filling it with paraffin wax.	 
<ul style="list-style-type: none">• Remove the paraffin wax from its plastic packaging.• Put the desired amount of paraffin wax (minimum 0.9 kg, maximum 1.35 kg) into the paraffin bath. In order to make it easier to put the paraffin wax into the paraffin bath, break up the blocks of paraffin wax. <p>⚠ Important Make sure that the paraffin bath is always filled with paraffin wax up to at least the MIN fill indicator line before use.</p>	
<ul style="list-style-type: none">• Insert the mains plug into the socket (220-240V, ~50/60Hz). The green operation light on the paraffin bath will light up. The device is now switched on. The red heating light on the paraffin bath will also light up, and will only go out when the specified heat setting has been reached.	
<ul style="list-style-type: none">• Turn the temperature controller to “hot” to melt the paraffin wax.	
<ul style="list-style-type: none">• Now place the lid on the paraffin bath and wait until the paraffin wax has completely melted (around 2-3 hours). <p>⚠ Warning If the paraffin bath is set to “hot”, never dip your hands, feet or elbows in the paraffin wax.</p>	

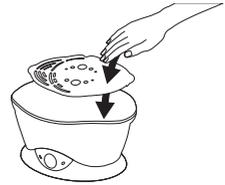
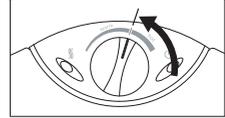
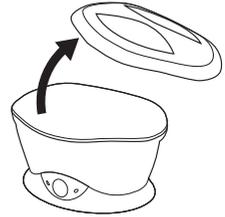
- As soon as the paraffin wax has completely melted, remove the lid.
- Turn the temperature controller to the marked line between “warm” and “hot”.
- Lay the protective grille included in the delivery contents in the liquid paraffin wax and wait until it has sunk to the bottom of the paraffin bath.
- Put the lid back onto the paraffin bath and wait approximately one hour, until the paraffin wax has cooled to an ideal operating temperature (60°C).

⚠ Warning

The room temperature can affect the power of the device and consequently the temperature of the wax.

Before submerging your hands, feet or elbows in the paraffin bath, use the inside of your wrist to check that the temperature of the wax is comfortable.

If a skin should form on the wax surface, increase the temperature by turning the temperature controller clockwise. If the paraffin wax is too hot for you, reduce the temperature by turning the temperature controller anti-clockwise.

7. Usage

⚠ Warning

Before submerging your hands, feet or elbows in the paraffin bath, use the inside of your wrist to check that the temperature of the wax is comfortable.

⚠ Warning

Take off your jewellery (e.g. watches, rings, etc.) before treatment.

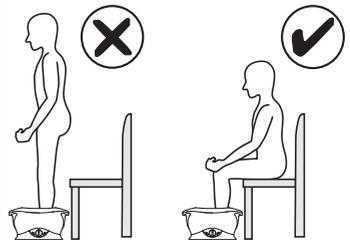
After you have completed the necessary preparations from “Chapter 6 Initial use”, you can begin using the device. To start the treatment, carry out the following steps:

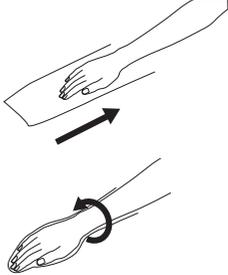
- We recommend applying a lanolin-based hand lotion to the areas to be treated before treatment. This will help you achieve a better result, and will make it easier to remove the paraffin wax after treatment.

⚠ Warning

When using the paraffin bath on your feet, only use it sitting down. Never stand in the paraffin bath.

- Briefly submerge the area to be treated (your hand, foot or elbow) for 1-2 seconds in the liquid paraffin wax. Remove your hand again and wait briefly until the wax has solidified. During this time, do not move the treated area, so that the wax can solidify properly.

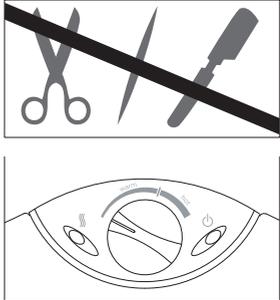
<ul style="list-style-type: none"> Repeat the process of submerging and pulling out six to seven times, until several layers of paraffin wax have formed over the treatment areas. If the paraffin wax is either too hot or too cold for you during the treatment period, turn the temperature controller clockwise or anti-clockwise, to increase or decrease the temperature as required. 	
<ul style="list-style-type: none"> Now put a sheet on the treated area, press it onto the paraffin wax and twist the end shut. Keep the sheet on for 20 minutes. <p>⚠ Warning If you are wearing the plastic sleeves on your feet, do not run around in them. You could slip.</p>	
<ul style="list-style-type: none"> After 20 minutes, take the sheet and the layers of wax off the treated area. Dispose the sheet in the general waste; it cannot be reused. 	
<ul style="list-style-type: none"> After each use, take the protective grille out of the paraffin bath. 	

8. Cleaning and maintenance

⚠ Important

If the device is only being used by one person, change the paraffin wax at regular intervals and clean the paraffin bath whilst doing so.

If the device is being used by several people, the paraffin wax should be changed before each treatment for reasons of hygiene.

<ul style="list-style-type: none"> Pull the mains plug out of the socket. Take the protective grille out of the paraffin bath. Allow the paraffin wax to solidify overnight. 	
<p>⚠ Warning Do not use any sharp or pointed objects to remove the wax block from the paraffin bath. You could damage or scratch the paraffin bath.</p> <ul style="list-style-type: none"> If the paraffin wax has solidified, press on one end of the wax block and lift it out of the paraffin bath. If the wax block cannot be removed easily, turn the temperature controller to “warm” for ten minutes. Then remove the mains plug and take the wax block out of the paraffin bath. 	

<ul style="list-style-type: none"> Then wipe the inside of the paraffin bath dry with a clean towel. <p> Important Never fill the paraffin bath with water in order to clean it.</p>	
<ul style="list-style-type: none"> After cleaning, store the paraffin bath in a cool, dry and dust-proof place. 	

9. Accessories and replacement parts

To purchase accessories and replacement parts, please visit www.beurer.com or contact the corresponding service address (according to the service address list) in your country. Accessories and replacement parts are also available from retailers.

Designation	Item and/or order number
2 x 450 g of paraffin wax + 30 plastic sleeves	589.42

10. What if there are problems?

Problem	Possible cause	Solution
The device is not working		For repairs, please contact Customer Services or an authorised retailer
Heating lamp/operation lamp does not light up	No power supply	Plug the mains adapter into the plug socket The mains adapter is faulty. Contact Customer Services or your retailer.
	LEDs have reached the end of their service life. LEDs faulty	For repairs, please contact Customer Services or an authorised retailer
Hard wax in paraffin bath	The oil in the paraffin wax is used up	Renew the paraffin wax. Only use genuine Beurer paraffin wax
	May occur due to frequent use	
	A paraffin wax with a higher melting point is used	
Discoloured paraffin wax	May occur due to frequent use	Renew the paraffin wax. Only use genuine Beurer paraffin wax
The paraffin wax takes longer to melt than usual	The paraffin wax is too cold	Store paraffin wax at room temperature
	The device is not working	For repairs, please contact Customer Services or an authorised retailer.

11. Disposal

Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any queries, please contact the appropriate local authorities.

12. Technical specifications

Model no.	MP 70
Voltage supply	220-240 V~, 50/60 Hz
Output	80 W
Protection class	II / □
Dimensions (L x W x H)	33 x 23 x 18.5 cm
Weight	1 kg

13. Warranty / Service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use. German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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