DE Massage-Fußwärmer Gebrauchsanweisung	2	RU Грелка для ног Инструкция по применению.....	50
EN Massaging foot warmer Instructions for use	10	PL But rozgrzewający z funkcją masażu Instrukcja obsługi.....	58
FR Chauffe-pieds massant Mode d'emploi.....	18	DA Massage-fodvarmer Betjeningsvejledning.....	66
ES Calientapiés con masaje Instrucciones de uso.....	26	SV Massagefotvärmare Bruksanvisning	74
IT Scaldapiedi con massaggio Istruzioni per l'uso.....	34	NO Massasjefotvarmer Bruksanvisning	82
TR Masaj ayak ısıtıcısı Kullanma Talimatı.....	42	FI Hierova jalkalämmitin Käyttöohje.....	90

DEUTSCH

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ENGLISH

Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, air and baby.

With kind regards
Your Beurer team


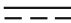
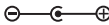
Contents

1. Included in delivery.....	12	5. Cleaning and maintenance.....	15
1.1 Unit description	12	6. Storage	15
2. Important instructions	12	7. Disposal.....	15
3. Intended use.....	14	8. What if there are problems?	16
4. Operation.....	14	9. Technical data	16
4.1 Initial use.....	14	10. Warranty / Service.....	16
4.2 Switching on	14		
4.3 Massage settings/heat settings.....	14		
4.4 Switching off.....	15		

Explanation of symbols

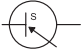The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Read the instructions!		Do not wash
	Do not insert pins!		Do not bleach
			Do not dry in a tumble dryer
	Do not use folded or rucked!		Do not iron
	Not to be used by very young children (0 - 3 years).		Do not dry-clean
	Only use in dry rooms	IPX1	Drip proof
	Dispose of packaging in an environmentally friendly manner		Manufacturer

	<p>This product satisfies the requirements of the applicable European and national directives.</p>		<p>The products demonstrably meet the requirements of the technical Regulations of the EAEU.</p>
	<p>The device has double protective insulation and therefore complies with protection class 2.</p>		<p>Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).</p>
	<p>Switching power supply</p>		<p>Energy efficiency level 6</p>
<p>IP 20</p>	<p>Protected against solid foreign objects 12.5 mm in diameter and larger</p>		<p>Direct current The device is suitable for use with direct current only</p>
	<p>Short-circuit-proof safety isolating transformer</p>		<p>Polarity</p>
	<p>The textiles used for this device meet the stringent human ecological requirements of Oeko Tex Standard 100, as verified by Hohenstein Research Institute.</p>		
	<p>WARNING: Warning of risks of injury or health hazards</p>		
	<p>CAUTION: Safety information about possible damage to appliance/accessories.</p>		
	<p>NOTE: Important information.</p>		


1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 Foot warmer with fleece lining
- 1 Mains adapter
- 1 Instructions for use

1.1 Unit description

- 1. Cable
- 2. Mains adapter
- 3. Control
- 4. Sliding switch for massage settings (0-1-2)
- 5. Sliding switch for heat settings (0-1-2)
- 6. Fleece lining
- 7. Foot warmer

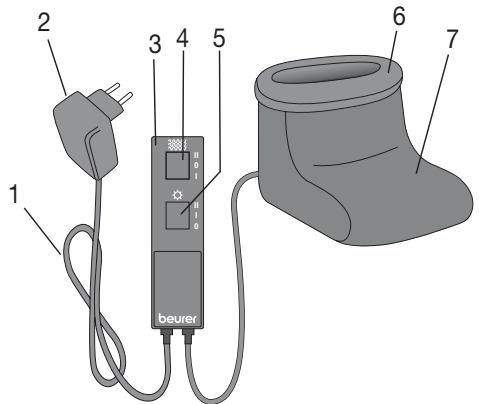2. Important instructions

Retain for future reference

WARNING

- Non-observance of the following notes may result in personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information is not only intended to protect your health and the health of others, it should also protect the product. For this reason, pay attention to these safety notes and include these instructions when handing over the product to others.
- This foot warmer must not be used by people who are not sensitive to heat or by vulnerable people who may not be able to react to overheating (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).
- This foot warmer must not be used by very young children (0-3 years old) as they are unable to respond to overheating.
- This foot warmer can be used by children older than 3 and younger than 8 years of age provided they are supervised. For this, the control must always be set to the minimum temperature.
- This foot warmer may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised and have been instructed on how to use the foot warmer safely, and are fully aware of the consequent risks of use.

- Children must not play with the foot warmer.
- Cleaning and user maintenance must not be performed by children unless supervised.
- This foot warmer is not designed for use in hospitals.
- This foot warmer is only intended for domestic/private use, not for commercial use.
- This foot warmer must not be used to keep animals warm.
- Do not use this foot warmer with shoes.
- Do not insert needles.
- Only use in dry rooms.
- Do not use if wet.
- Connect the unit only to the mains voltage listed on the type plate.
- Use only the original mains adapter provided (see “Technical data” chapter).
- The electrical and magnetic fields emitted by this foot warmer may interfere with the function of a pacemaker. However, they are still well within the limits: electrical field strength: max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 millitesla. Please consult your doctor and the manufacturer of your pacemaker before using this foot warmer.
- Do not pull, twist or make sharp bends in the cables.
- If they are not positioned properly, there may be a risk of becoming entangled in, being strangled by, tripping over, or stepping on the cable and control of the foot warmer. The user must ensure that excess lengths of cable, and cables in general, are safely routed.
- Please check this foot warmer frequently for signs of wear and tear or damage.

If any such signs are evident, if the foot warmer has been used incorrectly or if it no longer heats up, it must be checked by the manufacturer before being switched on again.

- Under no circumstances should you open or repair the foot warmer (including the accessories) yourself because faultless functionality can no longer be guaranteed thereafter. Failure to observe this will invalidate the warranty.
- If the mains connection cable of this foot warmer is damaged, it must be disposed of. If it cannot be removed, the foot warmer must be disposed of.
- When this foot warmer is switched on:
 - Do not place any sharp objects on it
 - Do not place any heat sources, such as hot water bottles, heat pads or similar, on it.

- The electronic components in the control warm up when the foot warmer is in use. For this reason, the control must never be covered or placed on the foot warmer when it is in use.
- It is essential to observe the information relating to the following chapters: Operation, Cleaning and maintenance, Storage.
- If you should have any questions about using our devices, please contact Customer Services.

3. Intended use

WARNING

This product is intended for warming and massaging human feet.

NOTE

This foot warmer is suitable for feet up to shoe size 47 (EU)/12.5 (UK).

4. Operation

4.1 Initial use

- Connect the mains adapter cable to the foot warmer control.
- Insert the mains adapter into the socket.

4.2 Switching on

- The fastest way to warm up the foot warmer is to initially select the highest heat setting (setting 2). The setting can be reduced later if necessary.
- The foot warmer can, of course, be used at any time, even when it is switched off: its fleecy fabric will warm the feet if there is no socket nearby.

4.3 Massage settings/heat settings

– Massage only:

Setting I (low): relaxing massage

Setting II (high): invigorating massage

The sliding switch for the heat settings is in the “0” position.

– Heat only:

Setting I: minimum heat

Setting II: maximum heat


The sliding switch for the massage settings is in the “0” position.

– Massage with heat:

Massage and heat setting freely selectable.

– No massage or heat:

Both sliding switches are set to “0”.

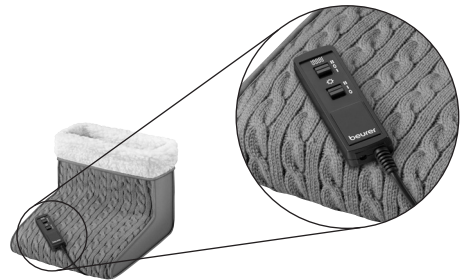WARNING

If the foot warmer is being used over several hours, we recommend that you select the lowest heat setting on the control to avoid overheating the feet, which may lead to burns to the skin.

4.4 Switching off

To switch the foot warmer off, set the sliding switch for massage and heat settings to the setting “0” (OFF).

NOTE

If the foot warmer is not in use, set the sliding switch to “0” (OFF). Disconnect the mains adapter from the foot warmer. Remove the mains adapter from the socket.

5. Cleaning and maintenance

WARNING

Before cleaning, always remove the mains adapter from the socket first. Then disconnect the mains adapter cable from the foot warmer control. Otherwise, there is a risk of damage.

IMPORTANT

The control must never come into contact with water or other liquids, as this may cause damage

- To clean the control, use a dry, lint-free cloth. Do not use any chemical or abrasive cleaning agents.
- Small marks on the foot warmer can be removed with a damp cloth and, if necessary, with a little liquid detergent for delicate laundry.

IMPORTANT

Please note that the foot warmer must not be washed, chemically cleaned, wrung out, tumble dried, put through a mangle or ironed. Failure to observe this may result in damage to the foot warmer.

- The removable fleece lining can be washed by hand.
- Use a delicate laundry detergent and measure it out according to the manufacturer’s instructions.

WARNING

Only use the foot warmer again once the foot warmer and fleece insert have fully dried. Otherwise, there is a risk of damage.

6. Storage

If you do not plan to use the foot warmer for a long period of time, we recommend that you store it in the original packaging.

IMPORTANT

- Allow the foot warmer to cool down before storing. Failure to observe this may result in damage to the foot warmer.
- To avoid damaging the foot warmer, do not place any objects on top of it whilst storing.

7. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

8. What if there are problems?

Problem	Cause	Solution
The foot warmer does not warm up or the massage function cannot be activated while: – the mains adapter is connected to a working socket. – the foot warmer control is connected to the mains adapter cable. – both sliding switches are set to setting 2 (heat + massage).	The foot warmer is faulty.	Send the foot warmer for servicing.

9. Technical data

Foot warmer technical data: See type plate on the foot warmer control.

For technical data about the mains adapter, see the type plate label on the mains adapter.

Permissible mains adapter:

EU mains plug: IVP1200-1500G

UK mains plug: IVP1200-2000B

Model identifier	IVP1200-2000B (UK mains plug)
Input voltage	100 V–240 V
Input frequency	50/60 Hz
Output voltage	12.0 V
Output current	2.0 A
Output power	24.0 W
Average efficiency during operation	86.5%
Efficiency load	84.8%
load power consumption	0.09 W

10. Warranty / Service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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