DE	Tageslichtlampe Gebrauchsanweisung	2	FI	Kirkasvalolamppu Käyttöohje.....	44
EN	Daylight therapy lamp Instructions for use.....	13	DA	Dagslyslampe Betjeningsvejledning.....	54
FR	Lampe de luminothérapie Mode d'emploi.....	23	NO	Dagslyslampe Bruksanvisning	64
SV	Dagsljuslampa Bruksanvisning	34	RU	Прибор дневного света Инструкция по применению.....	74

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Read these instructions for use carefully and keep them for later use; be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

With kind regards,
Your Beurer team

Why is the daylight therapy lamp called “Perfect Day”?

The beurer TL 45 Perfect Day daylight therapy lamp has three adjustable colour temperatures and therefore enables various applications in order to regulate the day-night rhythm.

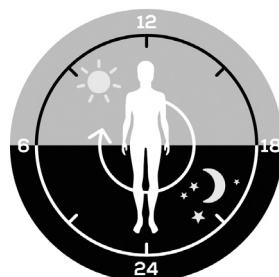
The daylight therapy lamp can either be used to imitate the progression of a sunny day or it can be used in a targeted manner in order to support the phases of concentration, therapy and relaxation.

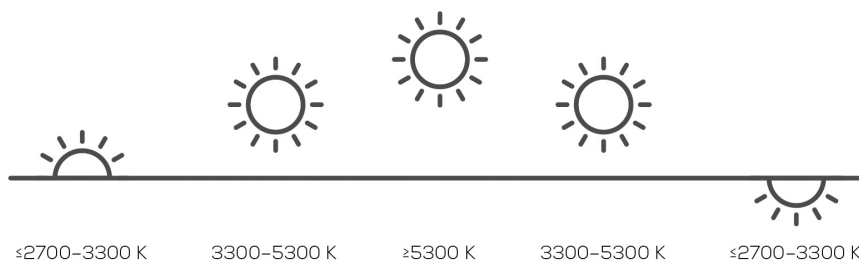
The TL 45 Perfect Day daylight therapy lamp is based on the concept of **“Human Centric Lighting”**.

In Human Centric Lighting (HCL for short), people and their perception of light are the focus. HCL deals with the psychological, physiological and psychobiological effect of light on people, for example light is the signal transmitter for the internal body clock.

HCL uses alternating colour temperatures and different levels of light intensity to help people concentrate or relax in the best possible way. Different colour temperatures have a different effect on the human eye, for example.

An increasing number of companies are creating the ideal workplace where employees are provided with the best possible light. This can increase motivation and productivity, and reduce errors and absences. The TL 45 Perfect Day daylight therapy lamp provides all of these benefits at your workplace or your home. It has three colour temperatures: “active”, “therapy” and “relax”.

	active	therapy	relax
Simulated time of day	Morning	Afternoon	Evening
Activity	Concentrated work	Therapy at 10,000 lux	Relaxation
Kelvin	4,300 kelvins	6,500 kelvins	3,000 kelvins
Lux	7,000 lux at a distance of approx. 25 cm	10,000 lux at a distance of approx. 20 cm	3,000 lux at a distance of approx. 20 cm

Why use a daylight therapy lamp?

When the hours of daylight are noticeably shorter in autumn and people increasingly stay inside in winter, the effects of a lack of light may become apparent. This is often described as “winter depression”. The symptoms can present themselves in a number of ways:

- Imbalance
- Low mood
- Lack of energy and listlessness
- Generally feeling under the weather
- Increased need for sleep
- Loss of appetite
- Difficulty concentrating

The cause of these symptoms is the fact that light – particularly sunlight – is essential for life and has a direct effect upon the human body. Sunlight indirectly controls the production of melatonin, which is passed to the blood only in darkness. This hormone shows your body that it's time to sleep. That's why more melatonin is produced in months with less sunshine, making it difficult to get up in the mornings because your body functions are powered down. Use the daylight therapy lamp immediately after waking up (i.e. as early as possible) to end the production of melatonin and to brighten your mood.

Lack of light also prevents the production of the happy hormone serotonin which is said to significantly influence our well-being. The application of light thus yields quantitative changes to hormones and neurotransmitters in the brain that have an effect on our activity levels, our feelings and our well-being. To compensate for such a hormonal imbalance, daylight therapy lamps can create a suitable replacement for natural sunlight.

In the medical field, daylight therapy lamps are used as compensation for the effects of a lack of light. Daylight therapy lamps simulate daylight over 10,000 lux. This light can influence the human body and be used as a treatment or as a preventative measure. Normal electric light, however, is not sufficient to influence the hormonal balance. This is because in a well-lit office, the light intensity is just 500 lux, for example.

Contents

1. Included in delivery.....	15	8. Cleaning and maintenance.....	19
2. Signs and symbols	15	9. Storage	19
3. Intended use.....	15	10. Disposal	19
4. Warnings and safety notes.....	15	11. What if there are problems?	20
5. Device description.....	17	12. Technical specifications	20
6. Initial use	17	13. Mains part	21
7. Usage	18	14. Warranty/service.....	22

1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.

- Daylight therapy lamp
- These instructions for use
- Mains part

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Read the instructions for use	 Note	Note on important information
 WARNING	Warning notice indicating a risk of injury or damage to health	 IMPORTANT	Safety note indicating possible damage to the device/accessories
	Manufacturer		Protection class II
	YYYYXWW / XXXXXX YYYYXWW = year of manufacture, year code and week of manufacture XXXXXX = serial number		The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices
	Disposal in accordance with the EC Directive on Waste Electrical and Electronic Equipment (WEEE).		Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
 Storage	Permissible storage temperature and humidity	 Operating	Permissible operating temperature and humidity
	For indoor use only		Dispose of packaging in an environmentally friendly manner
	Direct current		OFF/ON

3. Intended use

The daylight therapy lamp is intended for radiation on human bodies only. Do not use the daylight therapy lamp on animals. The device is only intended for domestic/private use. The device is not intended for commercial use. The daylight therapy lamp is intended to compensate the effects of lack of light, particularly sunlight, and provide relief from winter seasonal affective disorders, mood disorders and circadian phase sleep disorders.

4. Warnings and safety notes

WARNING

- The daylight therapy lamp is intended for radiation on human bodies only.
- Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.
- Ensure that the daylight therapy lamp is positioned on a stable base.
- The device must only be connected to the mains voltage that is specified on the type plate.
- Never submerge the device in water and do not use it in the bathroom.

- It must not be used on people with disabilities, children under 3 years of age or people with reduced sensitivity to heat (e.g. people with skin alterations due to illness), after taking pain relief medication, alcohol or drugs.
- People who do not exercise due care and attention should not use the product.
- Keep packaging material away from children (risk of suffocation!).
- If the device is warm, do not cover it or pack and store it.
- Always unplug the device and allow it to cool down before touching it.
- Do not touch the device with wet hands when it is plugged in and do not allow water to spray on the device. Operate the device only if it is completely dry.
- Ensure that you insert and remove the mains cable and the plug with dry hands only and that you touch the ON/OFF sensor with dry hands only.
- Keep the mains cable away from hot objects and naked flames.
- Protect the device from heavy impacts.
- Do not pull the mains part out of the socket using the mains cable.
- Do not use the device if it shows signs of damage or does not function properly. In such cases, contact Customer Services.
- If the mains cable of this device is damaged, the device must be disposed of.
- Disconnection from the power supply network is only guaranteed if have you unplugged the plug.
- Do not use the device in the presence of flammable anaesthetic gas connections with air, oxygen or nitrogen oxide.
- No calibration and no preventive checks or maintenance need to be carried out on this device.
- You cannot repair the device. The device contains no parts that you can repair.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of further use of the device.
- Store the cable out of the reach of small children, so as to prevent them from becoming entangled in the cable or from suffocating through strangulation.
- The device is intended for use by the patient themselves. The device and accessories are to be used in accordance with these instructions for use.
- Do not allow the product to fall from a height.
- Requirements for positioning the product:
 1. First, open the stand fully and set up the product in a horizontal position, to prevent it from tipping over.
 2. Do not shake the product when setting it up, to prevent it from falling.
 3. Do not touch the stand when setting up the product, to prevent it from falling.
 4. Do not pull on the cable of the mains part, to prevent the product from falling.

General notes

IMPORTANT

- Always consult a doctor before using the daylight therapy lamp if you are taking medication such as pain relief medication, medication to reduce high blood pressure or antidepressant medication.
- Diabetics and people who suffer from retinal diseases must be examined by an optician before using the daylight therapy lamp.
- Please do not use the device if you suffer from an eye disease such as cataracts, glaucoma, diseases of the optic nerve or inflammation of the vitreous body.
- Always consult a doctor before using the daylight therapy lamp if you have a strong sensitivity to light or you are prone to migraine attacks.
- If you have health concerns of any kind, consult your GP!
- Remove all packaging material before using the device.
- Light sources are excluded from the warranty.
- Switch-on test: after switching on the device, check if there are any bright flashes, dark areas / shadows or other anomalies. This is especially important to do once the lamp has reached the expected number of operating hours. If you notice anything unusual, please contact Customer Services.
- The mains part is part of the medical equipment.

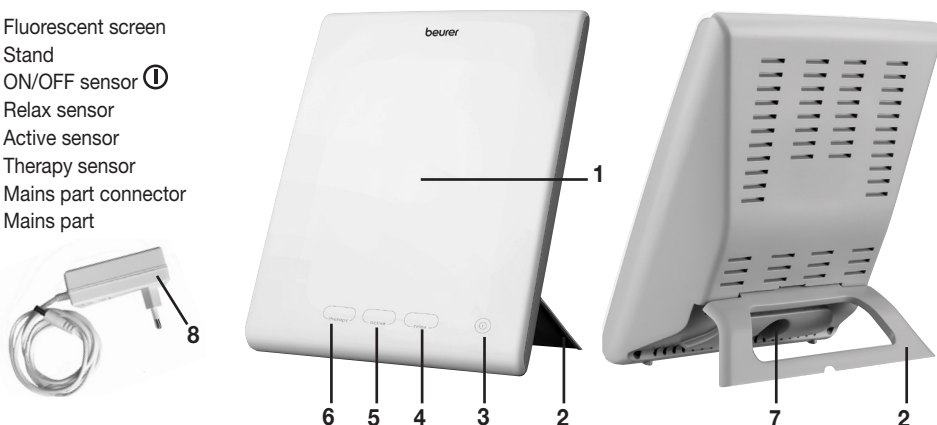
Repairs

IMPORTANT

- Do not open the device. Do not attempt to repair the device yourself. This could result in serious injury. Failure to comply with this instruction will void the warranty.
- For repairs, please contact Customer Services or an authorised retailer.

5. Device description

1. Fluorescent screen
2. Stand
3. ON/OFF sensor 
4. Relax sensor
5. Active sensor
6. Therapy sensor
7. Mains part connector
8. Mains part

6. Initial use

Take the device out of the plastic wrapping. Check the device for damage or faults. If you notice any damage or faults on the device, do not use it and contact Customer Services or your supplier.

Setting up the device

Place the device on an even surface. The position should be chosen to ensure a distance of between 20 cm and 45 cm between the user and the device. The daylight therapy lamp is most effective at this distance.

Note:

If the device has been in storage or recently transported, keep it for at least two hours at room temperature before using it.

Mains connection

IMPORTANT

To prevent possible damage to the device, the daylight therapy lamp must be used only with the mains part described here.

1. Insert the mains part into the mains part connector provided for this purpose on the rear of the daylight therapy lamp and into a suitable mains socket. The mains part must be connected only to the mains voltage that is specified on the type plate.
2. After using the daylight therapy lamp, unplug the mains part from the mains socket first and then disconnect it from the daylight therapy lamp.

i Note:

- Ensure that there is a mains socket close to the set-up area.
- Arrange the mains cable so that no-one will trip over it.

7. Usage

1	Plugging in the mains part Insert the mains part into the mains part connector on the rear of the daylight therapy lamp and into a suitable socket.																								
2	Switching on the daylight therapy lamp To switch on the daylight therapy lamp, touch the ON/OFF sensor ①.																								
3	Enjoying the light Select your desired setting by using the touch sensors on the daylight therapy lamp. We recommend the following two applications: Example of application in order to imitate a sunny day <table><tr><th>Programme</th><th>Start of application</th></tr><tr><td>relax</td><td>6 am</td></tr><tr><td>active</td><td>10 am</td></tr><tr><td>therapy</td><td>12 pm</td></tr><tr><td>active</td><td>2 pm</td></tr><tr><td>relax</td><td>8 pm</td></tr></table> Example of application for optimum light during a work day <table><tr><th>Programme</th><th>Start of application</th></tr><tr><td>therapy</td><td>6 am</td></tr><tr><td>active</td><td>8 am</td></tr><tr><td>relax</td><td>12 pm</td></tr><tr><td>active</td><td>1 pm</td></tr><tr><td>relax</td><td>8 pm</td></tr></table>	Programme	Start of application	relax	6 am	active	10 am	therapy	12 pm	active	2 pm	relax	8 pm	Programme	Start of application	therapy	6 am	active	8 am	relax	12 pm	active	1 pm	relax	8 pm
Programme	Start of application																								
relax	6 am																								
active	10 am																								
therapy	12 pm																								
active	2 pm																								
relax	8 pm																								
Programme	Start of application																								
therapy	6 am																								
active	8 am																								
relax	12 pm																								
active	1 pm																								
relax	8 pm																								
4	Therapy setting application Position yourself as close as possible to the daylight therapy lamp (approx. 20 – 45 cm). You can continue to go about your day-to-day activities during application, including reading, writing, making telephone calls, etc. • Keep looking directly into the light for a short period of time, as it is absorbed and shows an effect via the eyes/retina. • Use the daylight therapy lamp as often as you like. However, the treatment is most effective if you carry out the light therapy for at least 7 successive days according to the prescribed times. • Regardless of the recommendation in point 3, you can use the therapy application at different times. The most effective time of day for treatment is between 6 am and 8 pm. • Do not look directly into the light for the entire application period, as this may cause over-stimulation of the retina. • Start with a brief application period and gradually increase the time over the course of a week. Note: You may experience pain in the eyes and headaches after the first few applications. This pain should disappear in further sessions as the nervous system becomes accustomed to the new stimuli.																								

5	What you need to bear in mind		
	We recommend a distance of between 20 and 45 cm from the face to the daylight therapy lamp during application.		
	The duration of the application depends on the distance:		
	Lux	Distance	Application duration
	10,000	Approx. 20 cm	½ hour
6	5,000	Approx. 30 cm	1 hour
	2,500	Approx. 45 cm	2 hours
	In principle: The closer you are to the light source, the shorter the application time.		
7	Enjoying the light over longer periods		
	Repeat application on at least 7 consecutive days during darker periods of the year, or for longer depending on your individual needs. Application should take place during the mornings where possible.		
	Switching off the daylight therapy lamp		
	To switch off the daylight therapy lamp, touch the ON/OFF sensor ①.		
	The light will go out. Unplug the mains part from the mains socket.		
	 IMPORTANT The daylight therapy lamp remains hot after use. Allow the daylight therapy lamp to cool down sufficiently before putting it away and/or packing it away.		

8. Cleaning and maintenance

Clean the device from time to time.

WARNING

- Ensure that no water gets inside the device!
The device must be switched off, disconnected from the mains and allowed to cool down each time before cleaning.
- Do not clean the device in a dishwasher!
Use a slightly damp cloth to clean the device.
- Do not use any abrasive cleaning products and never submerge the device in water.
- Do not touch the device with wet hands when it is plugged in and do not allow water to spray on the device. Operate the device only if it is completely dry.

9. Storage

If you are not going to use the device for an extended period of time, disconnect it and store it in a dry place, out of the reach of children.

Follow the storage instructions provided in the “Technical specifications” chapter.

10. Disposal

For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with the EC Directive on Waste Electrical and Electronic Equipment (WEEE). If you have any questions, please contact the local authorities responsible for waste disposal.

11. What if there are problems?

Problem	Possible cause	Solution
Device does not light up	Device is switched off	Touch the ON/OFF sensor ①.
	No electricity	Plug a mains part into a suitable socket.
	No electricity	The mains part is faulty. Contact Customer Services or your retailer.
	LEDs have reached the end of their service life. LEDs faulty.	For repairs, please contact Customer Services or an authorised retailer.

12. Technical specifications

Model no.	GCE 503		
Type	TL 45 Perfect Day		
Dimensions (W, H, D)	200 x 250 x 60 mm		
Weight	Approx. 475 g		
Light	LEDs		
Maximum light output	therapy	25 W/m ²	
	active	25 W/m ²	
	relax	14 W/m ²	
Nominal output power	therapy	25 W ± 2 W	
	active	16 W ± 1 W	
	relax	6 W ± 1 W	
Light intensity	therapy	10,000 lux (distance of approx. 20 cm) 6,500 K	
	active	7,000 lux (distance of approx. 25 cm) 4,300 K	
	relax	3,000 lux (distance of approx. 20 cm) 3,000 K	
Radiation	Radiance output outside of the visible range (infrared and UV) is low enough that no eye or skin damage is anticipated.		
Operating conditions	0°C to +35°C, 15 – 90% relative humidity, 700 – 1060 hPa ambient pressure		
Transport and storage conditions	-20°C to +60°C, 15 – 90% relative humidity, 700 – 1060 hPa ambient pressure		
Threshold of short wavelength	therapy	445 nm	
	active	445 nm	
	relax	445 nm	
Operating height	≤2,000 m		
Product classification	External power supply, IP21		

Colour temperature of LEDs	therapy	6,500 K ± 300 K
	active	4,300 K ± 300 K
	relax	3,000 K ± 300 K
Expected service life of the device	10,000 hours	

Maximum radiance output of the TL 45 Perfect Day

Radiance output	Risk group classified in accordance with IEC 60601-2-57: 2011	Maximum value		
		Therapy	Active	Relax
EUVA: Eye UV-A	Exempt Group	$4.784 \times 10^{-5} \text{ W} \cdot \text{m}^{-2}$	$7.349 \times 10^{-6} \text{ W} \cdot \text{m}^{-2}$	$0 \text{ W} \cdot \text{m}^{-2}$
ES: Actinic UV skin & eye	Exempt Group	$7.249 \times 10^{-4} \text{ W} \cdot \text{m}^{-2}$	$2.205 \times 10^{-10} \text{ W} \cdot \text{m}^{-2}$	$0 \text{ W} \cdot \text{m}^{-2}$
EIR: Infrared radiation hazard exposure limits for the eyes	Exempt Group	$1.045 \times 10^{-1} \text{ W} \cdot \text{m}^{-2}$	6.252×10^{-2}	$7.208 \times 10^{-2} \text{ W} \cdot \text{m}^{-2}$
EH: Thermal hazard for the skin	Exempt Group	$2.487 \text{ e}^1 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$	$1.477 \text{ e}^1 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$	$6.182 \text{ e}^0 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$
LB: Blue light	Exempt Group	$3.094 \times 10 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$	$7.266 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$	$4.15 \times 10 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$
LR: Retinal thermal	Exempt Group	$3.918 \times 10^2 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$	$9.968 \times 10 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$	$7.563 \times 10 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1}$

Subject to technical changes.

The serial number is located on the device.

Brightness: 10,000 lux (this point about the light intensity is merely for the purpose of information. With regard to the standard IEC 60601-2-57, this light source is classified as an Exempt Group)

Notes on electromagnetic compatibility

The device complies with the EU Medical Devices Directive 93/42/EEC (therapy setting), the German Medical Devices Act (Medizinproduktegesetz) and the European standard EN 60601-1-2 (in accordance with CISPR-11, IEC61000-3-2, IEC61000-3-3, IEC 61000-4-2, IEC61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-8, IEC 61000-4-11) and is subject to particular precautions with regard to electromagnetic compatibility.

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

13. Mains part

Model no.	MP 45-240150-AG MP 45-240150-AB
Input	100–240 V ~ 50/60 Hz
Output	24 V DC, 1.5 A, only in conjunction with GCE503

Protection	The device is double-protected.
	Polarity of the DC voltage connection
Classification	IP21, protection class II
Manufacturer	Mitra Power Solutions Co., LTD
Weight	Approx. 170 g

14. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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Globalcare Medical Technology Co., Ltd.
Distributed by BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)
Donawa Lifescience Consulting Srl, Piazza Albania, 10, 00153 Rome, Italy